

## **OUR SYSTEM**

Our Equi'Nov kits have been designed to aid recovery after trauma or exercise.

They have an analgesic action and generate anti-hemorrhagic, anti-edematous and muscle-relaxing effects.

Our magnetic fabric reduces pain and swelling, while stimulating blood circulation. It promotes toxin elimination, healing and recovery. Finally, it helps prepare muscles and tendons for exertion.

Available in adjustable universal size.

The hock kit is sold individually.

- Place the kit in the freezer for at least 8 hours before use. However, if the kit is returned to the freezer immediately after a cryotherapy session, 2 h of freezing will be sufficient to prepare for the next session.
- A cryotherapy session should last 20 to 30 minutes maximum.
- Apply between 5 and 10 pressures with the pump, depending on the horse's feelings and preferences.

Our kits are a light cryotherapy device that allows access to hard-to-treat areas, such as deep tendon injuries or lacerations.

The combination of these effects makes our products an indispensable tool for faster recovery and easier, less painful rehabilitation.

