



MAGNETIC CRYOTHERAPY KIT: CRYO-GAITERS


3 in 1 effects:

- Cryotherapy
- Compression
- Magnetic field

OUR SYSTEM

Equi'Nov gaiters have been designed to aid recovery after trauma or exertion.

Our gaiters have an analgesic action and generate anti-hemorrhagic, anti-edematous and muscle relaxant effects.

Our magnetic fabric  reduces pain and swelling, while stimulating blood circulation. It promotes toxin elimination, healing and recovery. Finally, it prepares muscles and tendons for exertion.

Gaiters are sold in pairs.

- Place gaiters in the freezer for at least 8h before use. However, if the gaiters are returned to the freezer immediately after a cryotherapy session, 2 h of freezing will be sufficient to prepare for the next session.

- A cryotherapy session should last 20 to 30 minutes maximum.

- Apply between 5 and 10 pressures with the pump, depending on the horse's feelings and preferences.

Our Gaiters are a light cryotherapy device that allows access to hard-to-treat areas, such as deep tendon injuries or lacerations.


The combination of these effects makes our products an indispensable tool for faster recovery and easier, less painful rehabilitation.



Taille des Guêtres	S	M	L	XL
Taille en cm ©	30	34	38	42

Measurements are approximate, as each horse is different, the table is only to use as informative purpose.

Taille simplifiée	Poney	COB	Cheval	Cheval
Hauteur du canon	26 cm	28 cm	30 cm	36 cm
Tour du boulet	25 cm	30 cm	33 cm	36 cm
Antérieure	S	M	L	XL
Postérieure	M	L	XL	XL

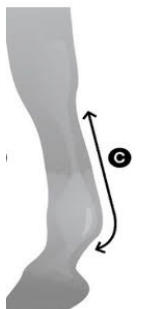



MAGNETIC CRYOTHERAPY KIT : STIFLE


3 in 1 effects:

- Cryotherapy
- Compression
- Magnetic field

OUR SYSTEM

Our Equi'Nov kits have been designed to aid recovery after trauma or exercise.

They have an analgesic action and generate anti-hemorrhagic, anti-edematous and muscle-relaxing effects.

Our magnetic fabric  reduces pain and swelling, while stimulating blood circulation. It promotes toxin elimination, healing and recovery. Finally, it helps prepare muscles and tendons for exertion.

Available in adjustable universal size.

- Place the cold packs in the freezer for at least 8 hours before use. However, if the cold packs are returned to the freezer immediately after a cryotherapy session, 2 h of freezing will be sufficient to prepare for the next session.

- A cryotherapy session should last 20 to 30 minutes maximum.

- Apply between 5 and 10 pressures with the pump, depending on the horse's feelings and preferences.

Our kits are a light cryotherapy device that allows access to hard-to-treat areas, such as deep tendon injuries or lacerations.

The combination of these effects makes our products an indispensable tool for faster recovery and easier, less painful rehabilitation.





MAGNETIC CRYOTHERAPY KIT : HOCK

3 in 1 effects:


- Cryotherapy
- Compression
- Magnetic field



OUR SYSTEM

Our Equi'Nov kits have been designed to aid recovery after trauma or exercise.

They have an analgesic action and generate anti-hemorrhagic, anti-edematous and muscle-relaxing effects.

Our magnetic fabric  reduces pain and swelling, while stimulating blood circulation. It promotes toxin elimination, healing and recovery. Finally, it helps prepare muscles and tendons for exertion.

Available in adjustable universal size.

The hock kit is sold individually.

- Place the kit in the freezer for at least 8 hours before use. However, if the kit is returned to the freezer immediately after a cryotherapy session, 2 h of freezing will be sufficient to prepare for the next session.

- A cryotherapy session should last 20 to 30 minutes maximum.

- Apply between 5 and 10 pressures with the pump, depending on the horse's feelings and preferences.

Our kits are a light cryotherapy device that allows access to hard-to-treat areas, such as deep tendon injuries or lacerations.

The combination of these effects makes our products an indispensable tool for faster recovery and easier, less painful rehabilitation.





MAGNETIC FABRIC


- Reduces pain and swelling.
- Stimulates blood circulation.
- Accelerates toxin elimination.
- Accelerates the healing and recovery process.
- Prepares muscles and tendons for exercise.



OUR MAGNETIC FABRIC

(Blanket and magnetic cryotherapy devices)

Our magnetic fabric is manufactured using technology derived from joint research into magnetic fields and mineralogy. The combination of minerals generates a magnetic field that acts directly on the pH of the water molecule, the main constituent of living matter.

Our magnetic fabric  reduces pain and swelling, while stimulating blood circulation. It promotes toxin elimination, healing and recovery. Finally, it prepares muscles and tendons for exertion.

Available in 3 sizes:

- 135 / 183 / 6.0
- 145 / 190 / 6.3
- 155 / 206 / 6.9

